

Meyland Carlift®



Unique to lift users from wheelchair into the car

The MEYLAND CARLIFT is the perfect lifting solution in the car! The MEYLAND CARLIFT lifts disabled persons in and out of cars without problems. The lift is operated either by the disabled person, or - if necessary - by an assistant.

Car use

The MEYLAND CARLIFT can be mounted in either the left or the right hand side of cars with two or four doors, and the lift arm is adjustable to fit the lift to different car sizes. The lifting/lowering function works by a 12 volt motor, which is supplied with power from the car battery. The lift is operated from a small control box with push buttons. The box is connected to the lift with a spiral cable long enough to make it possible for an assistant outside the car to operate the lift. The sideways movements are done manually. The MEYLAND CARLIFT is supplied with a nylon sling seat which has three mounting points to give a safe lift, and the sling seat is available in 2 different sizes (see backside).

When the MEYLAND CARLIFT is not in use or while driving it can be lifted off and put into the trunk.

Flexible

By using a removable installation in the door hinge MEYLAND CARLIFT can be used without leaving any holes or connection in the car.

Same lift / Many applications

This is an economical solution and the flexible MEYLAND CARLIFT can also be used inside the home. The MEYLAND CARLIFT is the perfect lifting solution at home!

The MEYLAND CARLIFT can also be used at home in the bedroom, the bathroom etc. and is operated in the same way as in the car. The lift can be used for lifting the person from the wheelchair to the bed, the toilet, the bath etc. Furthermore, the MEYLAND CARLIFT can lift a person off the


floor! For the use at home a height adjustable wall console is available, which is bolted onto the wall at the desired place.

Easy to handle

The MEYLAND CARLIFT is lifted into the console and supplied with power from a 12/230 volt transformer. Both lift and transformer can be used elsewhere in the house wherever wall consoles are mounted. An extension arm is available making the lift cover a wider area. The extension arm is also useful where the space is limited. The MEYLAND CARLIFT is lifted off the console in one single grip weighs only 13,5 kg – but lifts up to 125 kg

Meyland Carlift®

Sling seat for the Carlift

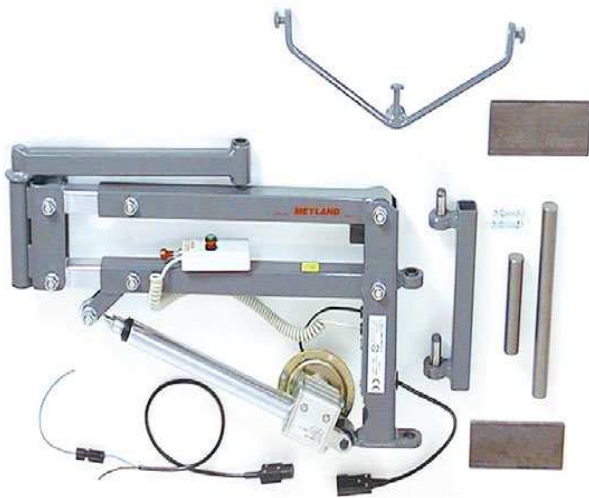

Tekniske data

Length	57-65 cm
Width	10 cm
Sling seat, standard	No. 5
Min. height:	35 cm
Lifting height:	39 cm
Max. load	125 kg
Weight:	13,5 kg

The sling seat is available in 2 sizes

Sling seat:	A	B	C	D
No. 1	113 cm	47 cm	89 cm	52 cm
Size corresponds approx. to a bodyweight of 70– 125 kg and a height of 1.70 m or more.				
No. 5 (std.)	82 cm	47 cm	85 cm	50 cm
Size corresponds approx. to a bodyweight of up to 70 and a height of less than 1.75 m.				

Accessories

- Extra lift console mounting set for a car
- Wall console
- Extension arm
- Height increasing pin
- Extra canvas sling seat

meyland-smith
mobility and more

Meyland-Smith A/S - Mads Clausens Vej 1 - DK-9800 Hjørring

+45 98 96 19 85 - Fax.: +45 98 96 19 86

www.meyland-smith.dk

E-mail: info@meyland-smith.dk