



Micro Voice amplifier

User manual

Art. Nr: 462230

EN 6.0



ABILIA

Table of Contents

Introduction	3
Overview of Micro Voice	3
Get started with Micro Voice	4
Charge Micro Voice	4
Use Micro Voice	5
Switch on (Power save)	5
Switch on (Constant).....	6
Switch off	6
Volume	6
Mute (standby).....	6
Maintenance.....	7
Clean Micro Voice.....	7
Troubleshooting.....	7
Storage and disposal.....	7
Storage	7
Disposal.....	7
Product specifications.....	8
Accessories	8
Important information.....	9
Safety Definitions	9
Safety instructions before using Micro Voice.....	9
Serious incidents	9
Conformance with requirements for medical devices	9
Product liability.....	10
Contact Information	11

Introduction

Micro Voice is a voice amplifier that supports those with a weak or strained voice, or those who regularly use their voice intensively.

This product is intended for indoor and outdoor use in a home, school or work environment.

Micro Voice is marketed as a technical aid for people with disabilities. It complies with all necessary requirements, regulations and directives for medical devices.

This use manual is intended for the end users. In this document you learn how to safely use Micro Voice.

Overview of Micro Voice



Micro Voice



Charger



Waist Belt



Headset



Volume Dial

Control Button



Microphone Port

Audio Output Port

LED Indicator



Charging Port

Charging LED

Get started with Micro Voice

Charge Micro Voice

⚠ WARNING: Only use the dedicated charger, supplied by Abilia, to charge the Micro Voice. Using a different charger may seriously damage the product.

1. Fully charge your Micro Voice before using it for the first time.
2. Connect the charger to the Charging Port.
3. Check that the charging LED flashes green to show it is working correctly.

The battery takes 6 hours to charge and lasts for about a week if used for 8 hours per day. The LED indicator can assume the following colours:



Green – Battery is charged. Green flashing – Battery is charging

Orange flashing – Battery is low

Red flashing – You have a short time before your battery runs out.


Use Micro Voice

⚠ WARNING: Do not put the microphone and the amplifier close together. This causes an acoustic feedback which, when set at high volume, may cause hearing impairment.


1. Decide how you want to wear your Micro Voice. You can attach it to your waist using the waist belt.
2. Plug the microphone into the Microphone Port.
3. Position the microphone:
 - a. For the AVI Headset Microphone: Fit it around the back of your head with the loops over your ears and move the microphone close to your mouth.
 - b. For the Lapel Clip Microphone: attach it to your clothing close to your mouth.
 - c. For the Transdermal Microphone: make sure it is fitted flatly against the skin at the top half of your neck above the collar line.



Headset

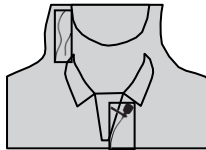

Lapel Clip Microphone

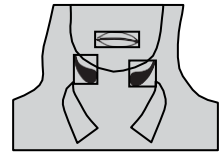

Transdermal Microphone

TIP! Transdermal Microphone: If you have a slim neck, tuck a small piece of fabric behind the back of the neckpiece to improve the fit. It should not feel so tight that it is uncomfortable.

Switch on (Power save)

1. Press the Control Button once.

You can start speaking into the microphone. The LED Indicator remains green while the Micro Voice is switched on. After 15 minutes of inactivity, the Micro Voice switches off.

Switch on (Constant)

NOTE! This feature is only applicable for devices with serial number 02969/VAMP, or newer.

1. Hold down the Control Button
The LED indicator flashes green and orange a few times and then turns red. The LED indicator then remains green while the Micro Voice is switched on. It remains on until it is manually switched off.

Switch off

1. Press and hold the Control Button until the LED indicator goes red.

Volume

1. Rotate the Volume Dial clockwise to increase the volume.
2. Rotate the Volume Dial counter-clockwise to decrease the volume.

Mute (standby)

If you would like to have a private conversation, you can put your Micro Voice on standby:

1. Press the Control Button once.
The LED indicator flashes orange to indicate that you are in Mute.
2. To return to normal operation press the Control Button once.
The LED indicator turns green.

Maintenance

Clean Micro Voice

Clean the amplifier with a slightly damp cloth. Clean the product when dirty or when preparing for reuse.

Troubleshooting

If you have questions regarding the functions or the settings in the product, speak to your product supplier or dealer.

When the product is not functioning correctly, contact Abilia's support, see the *Contact information*.

Storage and disposal

Storage

Keep the product indoors at room temperature and away from moist.

Disposal



This product contains electrical components that must be recycled at the end of the product's service life. Take it to an electrical waste recycling centre.

Product specifications

Dimensions, H × W × D	67 x 122 x 32 mm
Weight	210 g
Battery	Integrated rechargeable NiMH battery
Battery time	200h (standby mode) 40h (active mode)
Audio input and output	3.5 mm, stereo and mono

Accessories

It is possible to use alternative microphones for Micro Voice. The available microphones are listed as accessories to Micro Voice.


For more information about Micro Voice and the accessories, go to www.abilia.com/en/our-products/communication/rostforstarkare/micro-voice-amplifier or scan the QR code.



Important information

Safety Definitions


 **WARNING:** Risk of injury or death if the instructions are not followed.

 **CAUTION:** Risk of damage to the product if the instructions are not followed.

NOTE: Information that is necessary in a given situation.

TIP: Additional information that can improve the use of the product.

Safety instructions before using Micro Voice

 **WARNING:** Read the warning instructions that follow before you use the product.

- Before you use the product, read and understand the contents of this user manual.
- Use only the original battery charger that is supplied with the product to charge the battery.
- Do not put the microphone and the amplifier close together. This causes an acoustic feedback which, when set at high volume, may cause hearing impairment.

Serious incidents

If a serious incident related to the product occurs, report this to the manufacturer and the competent authority of your country. A serious incident directly or indirectly led, might have led or might lead to the death of a patient, user or other person; the temporary or permanent serious deterioration of a patient's, user's or other person's state of health; or a serious public health threat.

Conformance with requirements for medical devices

Micro voice amplifier is marketed as a technical aid for people with disabilities. It complies with all necessary requirements, regulations and directives for medical devices.

Product liability

The product is developed and risk-assessed according to ISO 14971. The manual and the technical specifications are made according to the safety assessments in the risk analysis. Always consider the safety when using the product. Abilia AB does not take responsibility for any consequences following incorrect use or installation of the product. Any form of damage or tampering with the product invalidates the warranty and disclaims the manufacturer's liability. The estimated lifetime of the product applies when used in accordance with its intended use and maintenance instructions.

Contact Information

Sweden

Abilia AB, Råsundavägen 6, 169 67 Solna, Sweden
Phone +46 (0)8- 594 694 00 | support@abilia.se | www.abilia.se

Norway

Abilia AS, Tevlingveien 23, 1081 Oslo, Norway
Phone +47 23 28 94 00 | support@abilia.no | www.abilia.no

United Kingdom

Abilia UK Ltd, Unit12 Buckingham Business Park, Swavesey,
Cambridge, CB24 4AE, United Kingdom
Phone +44 (0) 1954 281 210 | support@abilia.uk | www.abilia.uk

International

www.abilia.com/intl/get-products/distributors

