AbleNet®

Quick Start Guide BIGtrack Wireless

www.ablenetinc.com
(800) 322-0956

(11/19 Rev. A)

Connect to an iPad or iPhone

Requires iOS 13 or iPadOS 13 or newer

1. On the front of BIGtrack Wireless, select Bluetooth
2. On iPad or iPhone go to Settings App > Accessibility > Touch > AssistiveTouch > Turn AssistiveTouch on
3. On the iPad or iPhone Settings App > Accessibility > Touch > AssistiveTouch > Devices > Bluetooth Devices > Select your BIGtrack Wireless from the list of available devices
4. On the iPad or iPhone, select Pair
5. Once connected a circle cursor will appear on the screen of the device
6. Left-clicking will select items. Right-clicking will bring up the AssistiveTouch menu for additional actions.

Connect with Bluetooth

Compatible with most macOS, Windows, ChromeOS, and Android devices that have a Bluetooth receiver.

1. On the front of BIGtrack Wireless, select Bluetooth
2. On the device, go to the Bluetooth settings menu to connect a new Bluetooth device
3. Select your BIGtrack Wireless from the list of available devices

Connect with USB wireless receiver

Compatible with most macOS, Windows 10, ChromeOS, and Android devices that have a USB Type-A port.

1. Locate the USB wireless receiver inside of the battery compartment of BIGtrack Wireless.
2. Connect the USB wireless receiver to a USB Type-A port on your device
3. On BIGtrack Wireless, select the USB receiver mode

Batteries

BIGtrack Wireless requires 2 AA batteries. Batteries not included.

Accessibility switches for left and right-clicking

On the left and right side of BIGtrack Wireless are 1/8-in / 3.5-mm switch jacks. Connect any accessibility switch with a 1/8-in / 3.5-mm mono plug. Activating the accessibility switch will then activate a left or right-click.

For accessibility switches, visit www.ablenetinc.com.

Drag lock

Drag lock is a feature that enables a user to lock the mouse cursor onto an item and then drag that item without having to hold down the left click button.

We recommend turning drag lock off when using BIGtrack Wireless with an iPad or iPhone. When drag lock on BIGtrack Wireless is turned on, the user will experience a delay in the right-click on the iPad or iPhone.

To use drag lock:

1. On BIGtrack Wireless, turn drag lock to on
2. Connect BIGtrack Wireless to your device
3. On the device, position mouse cursor over the item you would like to move
4. Right-click two times within 1-second and the mouse cursor will lock onto the item
5. To release, single left or right-click

Pair Button

If BIGtrack Wireless is paired with a Bluetooth device that is not available and you need to pair it with a new Bluetooth device, hold the pair button down for five-seconds until the indicator light flashes three times. Turn BIGtrack Wireless off, then back on again to pair with a new Bluetooth device.

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Warranty & Contact Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com.

AbleNet, Inc. 2625 Patton Road, Roseville, MN 55113
(800) 322-0956 / www.ablenetinc.com

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FCC

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user’s authority to operate the equipment.

FCC logo

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between equipment and receiver.
- Consult AbleNet Technical Support or your local Distributor for help.

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FCC logo

For private households:

Information on Disposal of Users of WEEE

This symbol  on the product and/or accompanying documents means that

used electrical and electronic equipment (WEEE) should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product, to designated collection points where it will be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product.

Disposing of this product correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. Please contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

For professional users in the European Union:

If you wish to discard electrical and electronic equipment(EEE), please contact your dealer or supplier for further information.

For disposal in countries outside of the European Union:

This symbol is only valid in the European Union(EU). If you wish to discard this product please contact your local authorities or dealer and ask for the correct method of disposal.

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