# V2T-10

Voice to Text converter for landline phone  
Convertisseur Voix en Texte pour téléphone fixe  
Convertidor voz a texto para teléfonos de líneas fijas  
Sprache-zu-Text Umwandler für schnurgebundene  
Telefone

**English P1**

**Français P14**

**Español P28**

**Deutsch P41**

!!! Please refer to our website: [www.geemarc.com](http://www.geemarc.com) for an up to date user guide, as there may be important updates and changes you need to be aware of !!!

<b>CONTENTS</b> .....	<b>1</b>
<b>INTRODUCTION</b> .....	<b>2</b>
Unpacking the V2T-10 .....	2
<b>DESCRIPTION</b> .....	<b>3</b>
General Description .....	3
<b>INSTALLATION</b> .....	<b>5</b>
Setting Up – How to set the battery or AC-DC power adaptor ..	5
Setting Up – How to connect the V2T-10 to the Telephone .....	6
Setting Up – How to connect the V2T-10 to the Smart-Phone or Tablet .....	6
<b>USING THE V2T-10</b> .....	<b>7</b>
Turning the V2T-10 ON or OFF .....	7
Battery Indicator .....	7
<b>Voice to Text Converter</b> .....	<b>8</b>
- Working with Text Hear Personal Hearing Aid App on your Smart-phone or Tablet .....	8
- Select and Convert Voices to your Smart-phone or Tablet .....	9
- Adjust Voice Volume send to your Smart-phone or Tablet .....	9
- Mute Function during Voice to Text Converting .....	9
<b>TROUBLE SHOOTING</b> .....	<b>10</b>
<b>SAFETY INFORMATION</b> .....	<b>11</b>
<b>GUARANTEE</b> .....	<b>12</b>
<b>DECLARATIONS</b> .....	<b>13</b>

Congratulations on purchasing your Geemarc V2T-10. The V2T-10 is an assistive listening device working with **Text Hear Personal Hearing Aid** App running on your smartphone or tablet. It converts the speech to text and at same time you make/receive phone calls through your telephone line.

It is important that you read the instructions below in order to use your Geemarc V2T-10 to its full potential. Keep this user guide in a safe place for future reference.

## Unpacking the V2T-10

---

When unpacking the V2T-10, you should find the following in the box:

- V2T-10 Piece.
- 9V ALKALINE BATTERY.
- RJ9 TO RJ9 Cord.
- 3.5mm Plug to 3.5mm Plug Audio Cord.
- 1 User Guide


For product support and help visit our website at


**[www.geemarc.com](http://www.geemarc.com)**

Telephone **01707 387602**

Or fax **01707 832529**

## General Description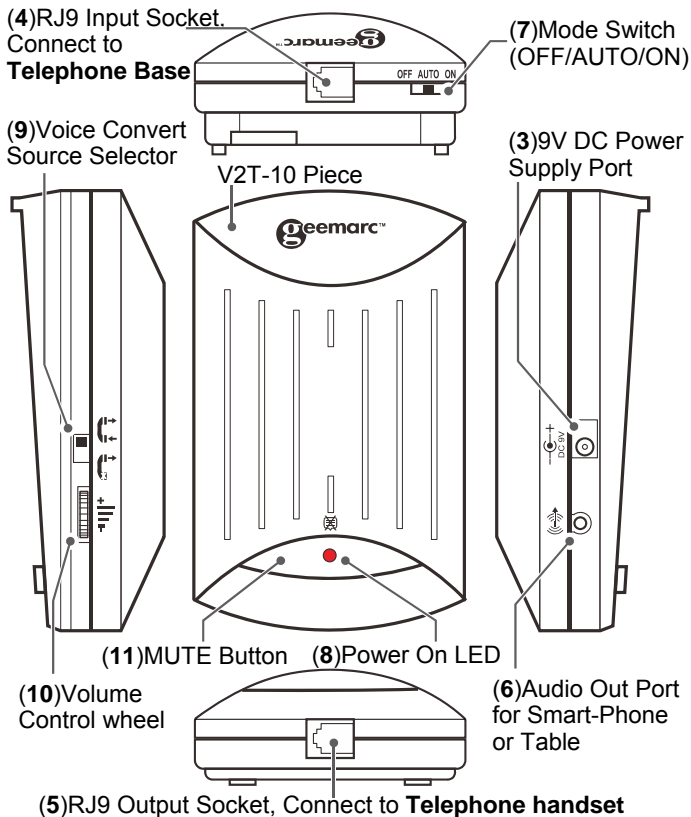

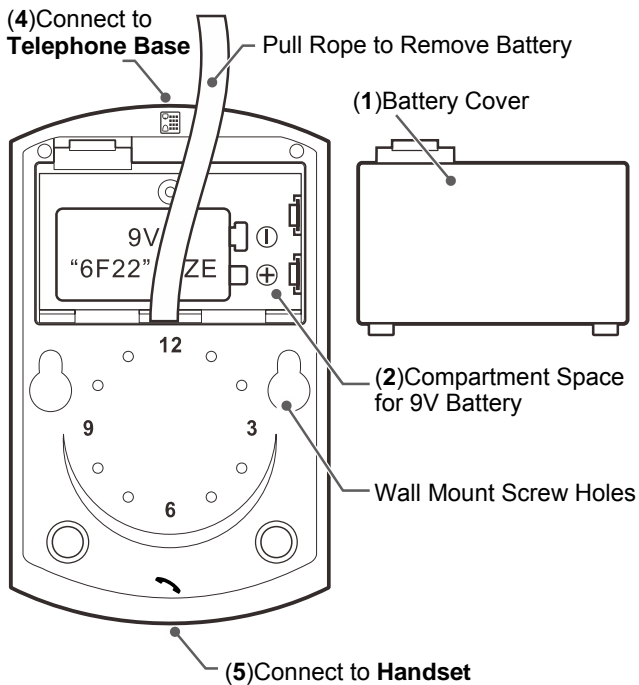
## Setting Up – How to set the battery or AC-DC power Adaptor


Open the battery compartment by pressing the small plastic lock on battery cover (1) located on the rear of the V2T-10 Piece. Insert the supplied battery (9V PP3) into the battery compartment (2), and then replace the battery cover.

**NOTE: Insert battery with the correct polarity as shown in the battery compartment.**

A 9V AC-DC Power Adaptor (not included) can be used to power the V2T-10 unit.

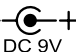Plug the Power Adaptor into the mains supply and plug the other end of the lead into the DC Power Supply Port (3) () on the right hand side of the V2T-10. Switch the mains supply on.

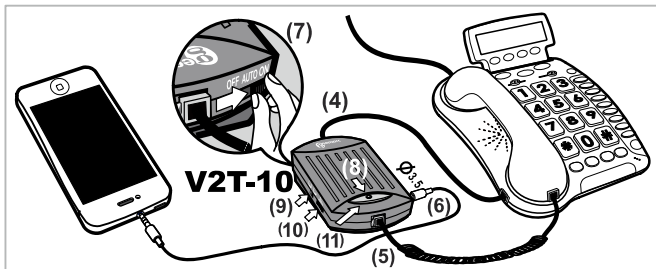**NOTE: Make sure the correct polarity of Adaptor output plug, centre is “+”.**

## Setting Up – How to Connect the V2T-10 to the Telephone

---

To connect your phone to the V2T-10:

- Unplug the Handset cord from your telephone base socket.
- Connect one end of RJ9 to RJ9 cord supplied with the V2T-10 to the telephone base socket and one end to the input socket **(4)** (  ) located at the top of your V2T-10 .
- Connect your handset cord to the output socket **(5)** (  ) located at the front of your V2T-10.

**IMPORTANT:** The V2T-10 cannot be used with one-piece telephones which have a keypad on the handset.

## Setting Up – How to Connect the V2T-10 to the Smart-Phone or Tablet

---

To connect your smart-phone or tablet to the V2T-10:

- Connect one end of 3.5mm to 3.5mm cord supplied with the V2T-10 to the 3.5mm socket of your smart-phone or tablet and one end to the Audio output port **(6)** (  ) located at the right hand side of your V2T-10.
- Download our **Text Hear Personal Hearing Aid** App from **Google play** and run it on your smart-phone or tablet.

**IMPORTANT:** At this stage, The V2T-10 only can be used with smart-phone or tablet running Android system.

## Turning the V2T-10 On and Off

---

Put the mode switch (7)  to **ON** position, The Power On LED (8) will display RED, and the V2T is ready to work.

To turn the V2T-10 off, put the mode switch (7) to **OFF** position. The RED LED will switch off.

**AUTO mode:** Put the mode switch (7) to **AUTO** position, The Power On LED (8) will display RED, if there has no phone call for a certain minutes, the V2T will auto power off to save battery life. When there is a phone call, the V2T-10 resumes automatically.

## Battery Indicator

---

The Battery indicator (8) should illuminate every time you use V2T-10. If the LED start to fade or does not light when your V2T-10 is in use, you should change the battery to ensure optimum performance.

## Making or Answering a phone call

---

When you want to make or answer a phone call, just do it as you do with your telephone!.


**IMPORTANT:** If you don't use the V2T-10 with Text Hear Personal Hearing Aid App, You can turn the V2T-10 off to save battery life. Turning V2T-10 to off will not affect your phone calls.

## Voice to Text Convertor


### - Working with Text Hear Personal Hearing Aid App on your Smart-phone or Tablet

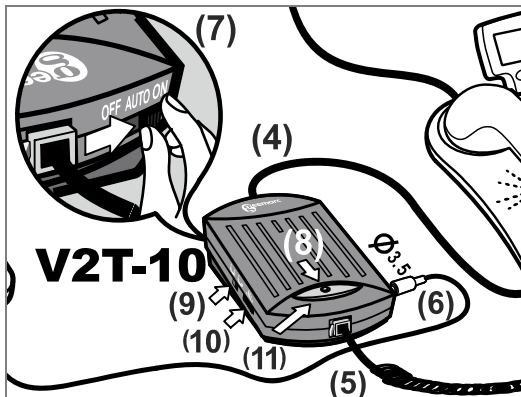- Download and run **Text Hear Personal Hearing Aid APP** on your smart-phone or tablet.
- Connect the V2T-10 to your smart-phone or tablet as instructed above.
- Put the V2T mode switch (7) (  OFF  AUTO  ON ) to **ON** or **AUTO** position.
- When you make, or answer a phone call, the speech will be converted to Text in **Text Hear Personal Hearing Aid APP** automatically.

## - Select and Convert Voices to your Smartphone or Tablet

---

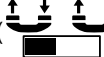You can convert **your voices to texts** () or **both you and friend's voices to text** () , using the voice convert source selector (**9**) () at the left side of V2T-10.

## - Adjust Voice Volume send to your Smartphone or Tablet

---

You can adjust the volume send to your Smart-phone or tablet by turning the wheel (**10**) () at the left side of V2T-10. Adjust the volume to get a best performance in the APP.

## - MUTE function during voice to text converting

---

If you don't want to convert the voice to text during phone call, just press and hold down the **MUTE button** (**11**) on the top of the V2T-10

**The V2T-10 appears not to be working**

- Make sure that the handset is securely plugged in.
- Make sure that the telephone base is securely plugged in.
- Make sure that the battery is inserted correctly and is of the recommended type.
- Make sure that the power supply cord is plugged in correctly i.e. into a working wall socket and into the V2T-10 Piece.
- Make sure the mode switch is set to ON or AUTO if using V2T to convert the voice to text.

**The Text Hear Personal Hearing Aid App appears not to be working correctly**

- Make sure that the audio cord is securely plugged in V2T-10.
- Make sure that the audio cord is securely plugged in Smart-Phone or tablet.
- Make sure that the volume is correctly adjusted.
- Make sure speech clearly when you talking in the phone.

## General

Do not open the unit. Contact the helpline for all repairs.

## Cleaning

Clean the V2T-10 with a soft cloth. Never use polishes or cleaning agents - they could damage the finish or the electrics inside.

## Environmental

Do not expose to direct sunlight.

Always ensure there is a free flow of air over the surfaces of the V2T-10.

Do not place any part of your product in water and do not use it in damp or humid conditions e.g. bathrooms.

Do not expose your product to fire or other hazardous conditions.

## GUARANTEE

---

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our helpline or visit our website at [www.geemarc.com](http://www.geemarc.com).

The guarantee does not cover accidents, negligence or breakages to any parts.

The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative.

The Geemarc guarantee in no way limits your legal rights.

**IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.**

**The guarantee applies to the United Kingdom only.**

## CE Directives

---

Geemarc Telecom SA hereby declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 2014/53/UE.

*The declaration of conformity may be consulted at [www.geemarc.com](http://www.geemarc.com)*

## Recycling directives

---

The WEEE (Waste Electrical and Electronic Equipment) Directive has been put in place so that products at the end of life are recycled in the best way.

When this product is out of order, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.
- Or hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.

For product support and help visit our website at

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